

SPOK21

How to open the threads for wheel chair adapter

Tools necessary:

1. drilling machine (small)
2. driller 1mm and 4,5mm

The wheelchair adapter is mounted with 4 screws at the bottom side of SPOK21. The thread for the screws are inside the case. They must be accessible by drilling holes in the case at the marked positions (4).

1. Switch off the SPOK21 at the bottom side.
2. At the bottom side you find 4 small marked positions. At this location please drill carefully holes with the 1,5mm driller, to have a better position for the 4,5mm driller.
Now drill with the 4,5mm driller carefully holes approx. 3mm deep until you meet the brass threads (you can hear it)
3. Remove the rest of the plastic from the thread. Attention: No metall parts should fall inside SPOK21.
4. Fix the adapter with the 4 screws at the now accessible threads.
5. Switch on the SPOK21 with the switch at the bottom.